

Please carefully read the information of use!

Arifusion eye drops

Instructions of use

Caution:

- Do not touch the pipette tip of the bottle to any surfaces, as this may lead to contamination of the solution.
- Do not transfer the solution to other bottles or containers.
- Do not use if the solution has changed color.
- If you feel discomfort in the eyes, tearing, change vision, redness of the eyes, stop using the product and consult your doctor because these symptoms can lead to serious diseases.
- Seek medical attention if swallowed.

Precautions :

- Always wash and dry your hands before using eye drops.
 - Close the bottle cap after use.
 - Keep the bottle tightly closed when not in use.
 - Do not store above 30°C.
 - Use before the expiry date specified on the packaging of the product.
 - Do not use more than 60 days after the first opening of the bottle.
 - Do not use if the bottle has been opened or damaged.
 - If more than one topical ophthalmic product is used, the drops should be administered at least fifteen (15) minutes apart.
-

Release form:

Eye drops. 10 ml , in a polyethylene (LDPE) bottle, with a built-in dropper and a polyethylene cap that is screwed on and with the control of first opening. 1 bottle in a pack.

Composition:

Each ml contains sodium hyaluronate 1 mg .

Excipients: sodium carboxymethylcellulose, glycerin, boric acid, sodium tetraborate, sodium chloride, potassium chloride, calcium chloride, magnesium chloride, sodium perborate, hydrochloric acid or sodium hydroxide, water for injections.

Properties:

Arifusion eye drops have a unique formula that successfully combines the action of three components: sodium hyaluronate, carboxymethyl cellulose and glycerin to lubricate, protect and restore the integrity of the eye surface.

Arifusion provides long-lasting eye protection and comfort with new Bridget Matrix technology for synergistic relief of dry eye symptoms.

Sodium hyaluronate is a natural polymer with viscoelastic properties that promote healing and prevent dehydration of the eye surface.

Carboxymethylcellulose is one of the strongest mucoadhesive polymers that maintains direct contact with corneal epithelial cells, providing long-lasting clinical benefits.

Glycerin in combination first provides lubricating and moisturizing effects on the surface of the eye, and then works under the tear film to protect the cornea.

Sodium perborate, a disappearing preservative, preserves the sterility of the solution in the bottle and disappears when instilled into the eyes, as it decomposes into oxygen and water, so it does not remain in the eyes.

Recommendations for use:

Eye drops are used for immediate relief of symptoms in all types of dry eye, and as protection against further irritation. Arifusion eye drops are compatible with all types of contact lenses.


It is necessary to consult with a doctor before use.

Method of application and dosage:

1-2 drops in each eye 3 times a day. After instillation, blink several times so that the solution is evenly distributed over the entire surface of the eye. When blinking, Arifusion eye drops form a protective membrane on the surface of the eye. If necessary, it can be used more often.

How to use Arifusion eye drops:

Remove the cap from the bottle. Hold the bottle upside down in one hand between the thumb and middle finger. Bring the tip of the bottle to your eye. Do not let it touch your eyes. Dispense one drop into the conjunctival sac formed between your eye and the eyelid by lightly tapping or pressing the base of the vial with your index finger (see picture). Do not squeeze the bottle.

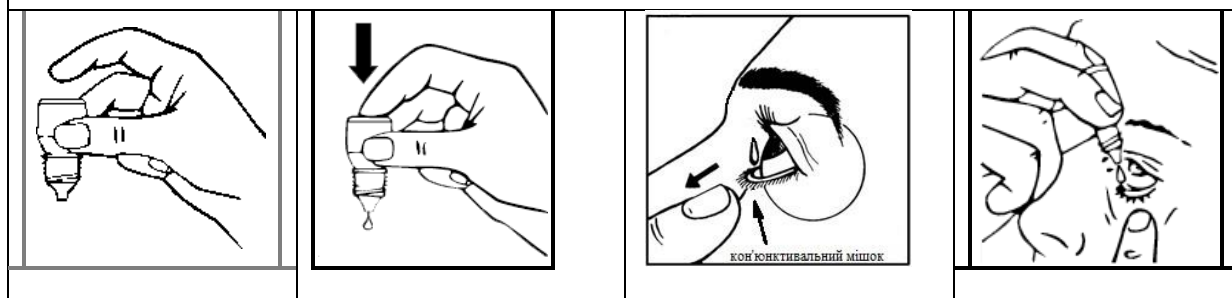

Adverse reactions:

In general, eye drops are well tolerated. In case of possible occurrence of adverse reactions associated with hypersensitivity to any of the components, consult with a doctor.

The ability to influence the reaction speed when driving a motor vehicle or working with other mechanisms: If the drops caused blurred vision, you should wait until the vision clears.

Use during pregnancy and lactation:

There is no information regarding the safety of use during pregnancy and lactation.

Contraindication:

Increased sensitivity to one of the components.

Storage conditions:

Store at a temperature no higher than 30°C and out of the reach of the children.

Expiration date:

2 years. Use within 60 days after first opening.

Packaging:

Each LDPE bottle contains 10 millilitres (ml) of sterile eye drops.

Manufacturer:

ARISTOPHARMA LTD.

7, Purana Paltan Line, Dhaka-1000, Bangladesh.

Unit 1: Plot No. 14-22, str. No. 11 & 12 Shampur Kadamtali I/A , Dhaka-1204, Bangladesh.

Name and location of the authorized representative in Ukraine:

LLC "Aristovision"

Kyiv-03022, Ukraine

prov. Kolomyyskyi 17/31a

office-1652

+38 044 257 32 57

e-mail: aristovision.office@gmail.com

The date of the last revision of the instructions for use:

25/03/2024



UA.TR.121

Markings are printed on the packaging:	
	Sterile. Sterilised using aseptic processing methods
	Read the instructions for use
	Batch code (lot)
	Date of manufacture (produced)
	Store at a temperature not exceeding 30°C
	Date of use (use by, month and last digits of the year)
	Manufacturer
	Mark of conformity with technical regulations
	Catalogue number
UA.TR.121	Identification number of the conformity assessment body