

Please carefully read the information of use!

Hypomer Gel ophthalmic gel

Instructions of use

Caution:

- Do not touch the tip of the tube to any surfaces, as this may lead to contamination of the gel.
- Do not use if the gel has changed color.
- If you feel discomfort in the eyes, tearing, change vision, redness of the eyes, stop using the product and consult your doctor because these symptoms can lead to serious diseases.
- Seek medical attention if swallowed.

Precautions :

- Always wash and dry your hands before using eye gel.
 - Close the tubes cap after use.
 - Keep the tube tightly closed when not in use.
 - Do not store above 30°C.
 - Use before the expiry date specified on the packaging of the product.
 - Do not use more than 60 days after the first opening of the tube.
 - Do not use if the tube has been opened or damaged.
 - If more than one topical ophthalmic product is used, the gel drops should be administered at least fifteen (15) minutes apart.
-

Release form:

Ophthalmic gel. 10 g in a tube with a screw cap. 1 tube in a pack.

Composition:

Each gram contains 3 milligrams (mg) of hypromellose (enriched with Carbomer 980).

Excipients: carbomer 980, sorbitol solution, disodium edetate, sodium perborate, hydrochloric acid or sodium hydroxide, water for injections.

Properties:

Hypomer Gel is a transparent colorless gel drops. The peculiarity of the product lies in its thixotropic properties, which ensure uniform distribution over the surface of the eye. Hypomer Gel combines the beneficial properties of gel drops to remain on the surface of the eye for a long time. This combination provides a long-lasting effect for users with eye discomfort.

The slight hypotonicity of the gel is quickly "neutralized" by the slightly hypertonic tear fluid caused by dry eyes.

Hypromellose protects the corneal epithelium, has a lubricating and softening effect, stabilizes, and reproduces the optical characteristics of the tear film.

Promotes epithelization, reduces hyperemia and the size of lesions (the effect is manifested after 3-5 days).

Sodium perborate, a disappearing preservative, preserves the sterility of the solution in the bottle and disappears when instilled into the eyes, as it decomposes into oxygen and water, so it does not remain in the eyes.

Recommendations for use:

For long-term relief of symptoms of dry eye-irritation, burning, tearing. Provides even coverage and a long-lasting effect.

It is necessary to consult a doctor before use.

Method of application and dosage:

Instil 1 drop into the conjunctival sac of the affected eye(s)

1-2 times a day. If necessary, it can be used more often.

Possible temporary blurring of vision after use, which quickly disappears with blinking.

How to use ophthalmic gel Hipomer Gel:

Remove the cap from the tube. Bring the tip of the tube to your eye. Do not let it touch the eye. Release one drop into the conjunctival sac formed between your eye and the eyelid. Blink several times to distribute the drops evenly over the surface of the eye.



Adverse reactions:

In general, Hypomer gel is well tolerated. Burning, allergic reactions or redness have been reported infrequently. With a severe form of dryness, burning may occur, which quickly passes.

In case of possible occurrence of adverse reactions associated with hypersensitivity to any of the components, consult a doctor.

Contraindication:

Hypersensitivity to any of the components of this product.

The ability to influence the speed of reaction when driving vehicles or working with other mechanisms: If the product caused blurred vision, you should wait until the vision clears.

Use during pregnancy and lactation:

There is no information regarding the safety of using the gel during pregnancy and lactation.

Storage conditions:

Store at a temperature no higher than 30°C and out of the reach of children.

Expiration date:

3 years. Use within 60 days after first opening.

Packaging:

10 g in a tube of sterile ophthalmic gel.

Manufacturer:

ARISTOPHARMA LTD.

7, Purana Paltan Line, Dhaka-1000, Bangladesh.

Unit 1: Plot No. 14-22, str. No. 11 & 12 Shampur Kadamtali I/A , Dhaka-1204, Bangladesh.

Name and location of the authorized representative in Ukraine:

LLC "Aristovision"

Kyiv-03022, Ukraine

prov. Kolomyyskyi 17/31a

office-1652

+38 044 257 32 57

e-mail: aristovision.office@gmail.com

The date of the last revision of the instructions for use:

25/03/2024



UA.TR.121

Markings are printed on the packaging:	
	Sterile. Sterilised using aseptic processing methods
	Read the instructions for use
	Batch code (lot)
	Date of manufacture (produced)
	Store at a temperature not exceeding 30°C
	Date of use (use by, month and last digits of the year)
	Manufacturer
	Mark of conformity with technical regulations
	Catalogue number
UA.TR.121	Identification number of the conformity assessment body